

Remodel Your Skin Naturally with SkinPen® Precision Elite



What is SkinPen® Precision Elite?

SkinPen® Precision Elite is a leading skin remodelling device that creates temporary small microscopic channels in your skin.

This simple procedure activates your body's natural skin-healing power to increase collagen so your skin returns to a healthier looking and youthful appearance.

Watch your skin look more youthful and rejuvenated with SkinPen® Precision Elite

Looking for Better Looking Skin Today?

Wrinkles and stretch marks

Do you want to improve the appearance of your fine lines and wrinkles? What about your stretch marks?

Acne & surgical scars

Do you want to get your self-confidence back by improving the appearance of unsightly scars?

Hyper & hypopigmentation

Pigmentation conditions can be emotionally devastating for many sufferers. SkinPen® Precision Elite may be the ideal treatment to improve your skin's appearance.

New Collagen, New You

How it Works:

Why does the media talk about collagen?

Why it is so important in medical aesthetics and skin rejuvenation? Collagen and elastin are the essential protein fibers in the skin that give it support and elasticity.

As we age, those fibers break down leading to fine lines, wrinkles, sagging skin, and large pores.

Regenerating collagen can also help in getting a more youthful appearance overall.

Collagen plays a key role in scars, such as those formed from acne or surgery. Improper signals in pigmentation lead to dyschromia (Hyper or hypopigmentation conditions). Skin remodelling can have an important role in the improved appearance of these conditions.



Simply Activate Your Own Collagen for More Youthful Looking Skin

SkinPen® Precision Elite is simple, comfortable and effective for skin rejuvenation in as little as 30 minutes.

What You Can Expect

Before treatment

The provider will gently cleanse the skin. Topical anesthesia may be administered to ensure maximum comfort throughout the treatment. Avoid sun exposure or sunburn and discontinue the use of topical retinoids at least 24 hours prior to procedure.

During treatment

Once the numbing cream has taken effect and has been completely removed from the skin, the provider will gently pass SkinPen® Precision Elite over the treatment area. As the tiny needles create millions of controlled micro-channels in the skin, you may feel light pressure and a mild buzzing sensation, with minimal discomfort.

These micro-channels stimulate your skin's natural wound-healing cascade, helping to remodel the tissue and support the formation of new collagen. The treatment lasts approximately 30 minutes.

After treatment

Most patients notice slight redness in the treated area, similar to a mild sunburn. Additional common treatment responses may include dryness, tightness and peeling. This is normal and should subside within 1 to 6 days.

A soothing hydrogel will be applied to the treated area to help reduce friction, support healing and maintain comfort following the procedure. Your provider may recommend a course of treatments based on your individual goals, and maintenance of your procedure may require multiple sessions. It is essential to follow all post-treatment instructions provided by your healthcare professional to support proper healing.

Let Your Body's Own Collagen Do the Job Naturally!



All natural:

Jump-start your new collagen with no heat, no chemicals, or invasive injections. Get beautiful and more youthful-looking skin.



Comfortable:

Light numbing cream may be used to assure maximum comfort.



Safe and effective:

Proven results from a treatment you can trust.



Quick and easy:

Get back to your life quickly with little to no downtime.



For all skin types year-round.